# Ikegami

**MKC-500HD** 

3CMOS FULL HDTV Medical Grade Camera High-end model



1920 X 1080

The best quality of Full HDTV camera for the medical application



The newly developed image processing capability achieves to express the more details and deeper focal depth compared to conventional cameras. Ikegami MKC-500HD is the most suitable camera for the low illumination surgery and fluorescence imaging with Full HDTV image quality.

: 1000 TV lines Horizontal Resolution, 54dB High S/N Ratio Resolutions

■ High sensitivity: 2000 lx/F12

# As natural as possible

### Full (1920x1080) HDTV

Internal signal processing of full HDTV Input with full HDTV Output. Various HDTV outputs are available, namely, 720p, 1080i and 1080p.

#### 1000 TV lines Horizontal Resolution

The latest 2 Mega pixels 1/3" CMOS sensors are employed to achieve the high resolution as HDTV images.

# **High Sensitivity**

The sensitivity is approximately 40% higher than the standard model and it can perform better even at the low illumination surgery.

### 54dB High S/N Ratio

Optimised signal processing circuitry achieves a high Signal-to-Noise Ratio of 54dB.

### **Digital Processing**

Video signal processing is done in the same signal process ASIC as Ikegamis' HDTV broadcast cameras to constantly achieve the same high level of quality and reliability.

# Image Processing Capability

Ikegami has newly developed the image processing capability to express the more details and deeper focal depth compared to conventional cameras.

# **Ultra Compact Camera-head**

The new optics for 3CMOS sensors is designed and the lens mount is C mount.





# A variety of functions for the best image quality

#### Automatic Iris Control

Accurate iris control is possible with auto-iris camera adaptors and Auto-iris lenses.

# Automatic Electronic Shutter System (Shutter Iris)

Electronic shutter (Shutter Iris) functionality to maintain the optimized performance in a variety of situations.

#### Auto Gain Control


The gain can be adjusted automatically to the proper brightness when the brightness it not enough.

# Slow Shutter (Signal Accumulation) Function

The slow shutter function can be useful at the low light application such as fluorescence.

#### Video Level Indicator

Level Indicator is provided on the front panel of CCU (camera control unit) for ease of operation.

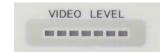

#### Skin Detail Enhancer

Fine control of human image colour detail is provided as standard to reproduce fine true to life skin tone.



# **DVI-D Output**

The camera can output at 1920x1080i/p 50/59.94Hz and 1280x720p 50/59.94Hz. This enables the camera to be used with external High Quality displays and PC monitors.

# Standard Output of SDTV (NTSC/PAL) **Down-converted Video**

The camera has an internal down-converter to output SDTV. Composite signal and S-VIDEO outputs are provided as

It can be used for the integration with existing equipment such as DVD recorders.







letter box



Medical Grade Camera High-end model



1920 X 1080




# Advanced Function to help operations

### Flip and Turn Function

Digital capture means that captured images can be flipped horizontally and vertically. Allowing for correction of image inversion from the optics used.

An optional foot pedal is required to allow an external operation of this function.



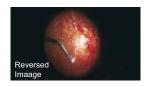

#### Scene File

4 scene files are provided to keep pre-set memories for functions and controls.

# Save and Load Function of Scene File

The set scene files can be saved into and loaded from the USB memory. When the camera setting is changed, it can be easily set back to the original settings.



#### **Picture Freeze**

Pressing the Freeze button on the CCU panel will hold the current image. This allows the detailed inspection of situations where the motion could hide the details.



This facility can also be synchronised with an external trigger for use in strobe applications.

# Suitable to the advanced surgery with the fluorescence application

# Fluorescein Mode


The fluorescein mode is available for the fluorescein imaging.

\* The combination with surgery microscopes for fluorescence optics is necessary for the fluorescein imaging.

# **ICG Mode**

The ICG mode is available for ICG imaging for NIR range.

- \* The combination with surgery microscopes for fluorescence optics is necessary for the ICG imaging.
- \* The camera should equip the exclusive optics as a factory option and need the combination with CVA-300R microscope adaptor.
- \* The visualization level should depend on the environment such as illumination, dye quantity, reaction with cells and so on.

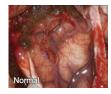



# A variety of useful features

# **User Friendly Operation**

The camera-head and CCU (camera control unit) are separated for the user friendly operation. The adjustment of the image and the settings of the functions can be operated at CCU even during the surgery not to disturb surgeons.

## **Camera Cable**

The cable connecting the camera-head and CCU(camera control unit) is available with 5m and 10m single and 5+10m combination cable.

# Still Image Capture

Still images (up to 1920x1080) can be captured and saved to the USB memory (JPEG or BMP). Controllable from external inputs such as a foot pedal.



#### **Remote Control Functions**

It can be used with external triggers such as the optional foot pedal, and programmed to control Image Flip/Turn, Still Image Capture, or Scene Files

# User Friendly Operation as Ikegami MKC Camera Series

The front panel and setting menu are followed commonly to Ikegami MKC camera series.

# Conformity to Safety Regulation and EMC Requirement for Medical Use

Medical Grade Safety Approval: UL60601-1, CSA

C22.2No.601.1, EN60601-1

EMC Approval: FCC Class A VCCI Class A

#### **Anti-biotic Coating**

The camera-head and CCU (camera control unit) utilises an anti-biotic coating to help to improve infection control.



# **Optional Accessories**

#### **Camera Cable**

5m single, 10m single and 5+10m combination camera cable





# Microscope Adaptor CVA-300RI

C-mount adaptor for a surgery microscope with the high quality optical lens for HDTV imaging.





### **Dual Foot Pedal**

Functions like Still Image Capture, Scene File, Flip and Turn Function can be assigned to this unit.







#### ■Rating

2 ch
1 ch
1 ch
1 ch
ninal 1 ch
1 ch
en optically.
nera cable
_

#### ■Performance

Resolutions	Horizontal: 1000 TV lines (1080i 50 or 59.94Hz)
S/N Ratio	54dB (Gamma/Detail off, Y signal)
Sensitivity	F12 at 2000lx/3200°K or better
Registration Error	Within 0.05% of on full screen

#### **■**Function

Auto White Balance
Auto Iris Control (DC Iris)
Auto Gain Control
Auto White Shading
Auto Shutter Iris
ON/OFF, and level adjustable
Up to 1/4 seconds slow shutter speed available
R-channel / B-channel (Level adjustable)
Exposure level adjustment
Built-in, Seven (7) steps
Built-in
Selectable in Menu
Image Flip and Turn available
The position of the output images can be adjusted in the horizontal (H) and vertical (V) position.
4 scene files
The scene file data can be saved into and loaded from an USB memory.
Pressing the Freeze button on the CCU panel holds the current image.
Still images (up to 1920x1080) can be captured and saved to the USB memory (JPEG or BMP).
ICG fluorescence mode available  **The camera should equip the exclusive optics as a factory option and need the combination with CVA-300R microscope adaptor.
Fluorescein mode available


 $<sup>\</sup>label{eq:combination} \mbox{\ensuremath{\mbox{\sc W}}\xspace{0.05em} The combination with surgery microscopes for fluorescence optics is necessary for the ICG and fluorescein$ 

#### ■Standard Accessories

AC Power Cable, Operation manual

#### ■Optional Accessories

5m single, 10m single and extension 10m cable available

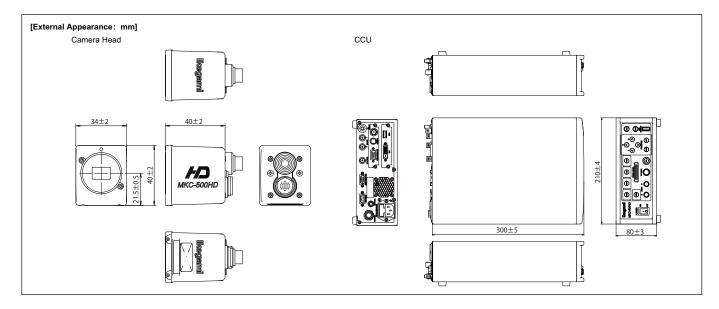

Design and specifications are subject to change without notice.

### Ikegami Tsushinki Co.,Ltd.

■ URL: http://www.ikegami.co.jp/en/

■ Head Office : 5-6-16 lkegami, Ohta-ku, Tokyo 146-8567, Japan · · · · · TEL.03-5700-1111 / FAX.03-5700-1137

■ Beijing Office: No.417 Jia Tai International Mansion, No. 41 East Fourth Ring Road, Chaoyang District, Beijing, China · · · · · · · TEL.010-85711804 / FAX.010-85711804



U245B132-ED





■ URL : http://www.ikegami.com

Ikegami Electronics (Europe) GmbH ■ URL: http://www.ikegami.de

■ U.K. Branch: Unit E1 Cologne Court, Brooklands Close, Windmill Road, Sunbury-on-Thames, Middlesex TW16 7EB, England · · · · · · · TEL.01932-769700 / FAX.01932-769710